



THUASNE

**MOBIDERM
& VENOFLEX
MEDICAL
COMPRESSION
SOLUTIONS**



PROUDLY DISTRIBUTED
IN NEW ZEALAND BY

medivex
HEALTHCARE LIMITED

Medivex Healthcare Ltd
335 Onehunga Mall,
Onehunga, Auckland 1061
0800 486 483
sales@medivex.co.nz
medivex.co.nz

The **reduction** phase

The reduction phase is also known as the intensive decongestive treatment.

Its goal is to achieve maximum reduction of the edema to come as close as possible to the volume of the opposite healthy limb.

The main treatments used are:

- **Manual lymphatic drainage (MLD)**, carried out by a specifically trained physiotherapist, to stimulate and facilitate the circulation of the lymph by promoting the unblocking of the lymph nodes.
- **Reducing and draining bandages** requiring the use of multi-component bandages which can be adapted to the volume and repositioned at will.
- **Coached physical activity** under bandages to improve lymph circulation, regain mobility and flexibility.
- **Skin care** to prevent complications related to lymph accumulation.

This phase may vary from one to several weeks, depending on the severity of the lymphedema, and can be renewed later.



Thuasne Protocol

The reduction phase is usually performed using multi-component bandages⁽¹⁾

The advantage of bandages is that they can be repositioned over and over to ensure the best possible fit to successive variations in edema volume.

There are different sorts of bandages, and it is their combined use that reduces edema.



1 PROTECT THE SKIN PROTECTIVE COTTON BANDAGE

In direct contact with the skin, this tubular cotton bandage is used to protect the skin. Made of cotton, it provides soft and lined protection of the skin weakened by the lymphedema.



2 MOBILIZE THE EDEMA MOBIDERM Bandage

Used under a reducing bandage, the Mobiderm technology mobilizes edematous or indurated subcutaneous tissue.



3 CONTAIN THE EDEMA BIFLEXIDEAL Short-stretch bandage

A short-stretch bandage (< 100% stretch) as the third layer helps containment of the edema, as well as drainage in case of physical activity (high working pressure)⁽²⁾.



4 COMPRESS THE EDEMA BIFLEX Graduated elastic compression bandage

A long-stretch bandage as last layer applies constant pressure, even at rest.

BIFLEX 16+ (light) is recommended for mild to moderate lymphedema.

BIFLEX 17+ (strong) is recommended for moderate to severe lymphedema.



Scan to learn how to bandage with MOBIDERM
<https://www.youtube.com/watch?v=7-wkbmFEkkk>



Compression bandages



Cutaneous protection

PROTECTIVE COTTON BANDAGE*

Protective bandage

Tubular bandage. Composition 100% cotton.
Protection of skin surface under non-adhesive bandage.

Width	Length	Color	Reference
11 cm	4 m	White	010101 01140000

MOBIDERM

MOBIDERM PADS

Dimensions	Designation	Reference
25 cm x 1 m	Small blocks 5 mm x 5 mm	371001 03310001
20 cm x 1 m	Big blocks 15 mm x 15 mm	371001 03310002

MOBIDERM BANDAGES

Dimensions	Designation	Reference
10 cm x 3 m	Small blocks 5 mm x 5 mm	372001 01030001
10 cm x 3 m	Big blocks 15 mm x 15 mm	372001 01030002

Containment

BIFLEXIDEAL

Short-stretch compression bandage

A short-stretch bandage (<100% stretch) as the third layer helps with containment of the edema, as well as drainage in case of physical activity (high working pressure).⁽¹⁾


Width	Length	Color	Reference
3 cm	5 m	White	110002 00350000
8 cm			110002 00850000

BIPLAST

Adhesive short-stretch compression bandage

Herringbone adhesive application. Multidirectional adhesiveness right to the edge of the bandage. Flexible and adjustable. 50.2% maximum stretch.⁽²⁾

Width	Length	Color	Reference
3 cm	2.5 m	White	200001 00325000
10 cm			200001 01025000

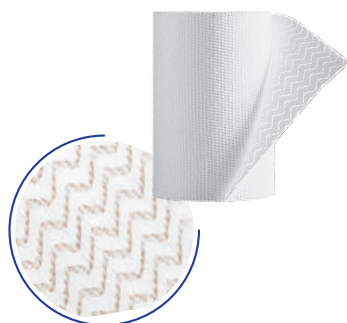

* Medical Device (MD) accessory. ** Contains also polyamide and elastane

(1) Quéré - Prospective multicenter observational study of lymphedema therapy: POLIT study - Vascular Disease Journal - 2014 - Pages 4 and 5

(2) Test report « Cyclage à force donnée » NF14704-1

Compression bandages



Compression



BIFLEX+ GRADUATED

Graduated elastic compression bandage

Two-ways elastic bandage. Elastic in length and extensible in width. Repeatable control of the tension exerted thanks to the fitting indicator. Multiple overlappings possible to achieve multiple pressure level.




2 versions

	Width	Length	Color	Reference
BIFLEX 16+ (Light) 	8 cm	5 m	Beige	160002 00850002
BIFLEX 17+ (Strong) 	8 cm	4 m	Beige	170002 00840002
	10 cm	3 m	Beige	170002 01030002



BIFLEX application in 2 simple steps

1/ STRETCHING

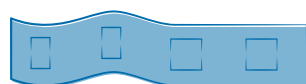


Rectangle → Square

2/ OVERLAPPING

BIFLEX 16+

Pressure □□□■ Pressure □□□■

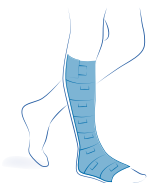


Half-square
Overlapping




Full-square
Overlapping

BIFLEX 17+

Pressure □■□■ Pressure ■□□■

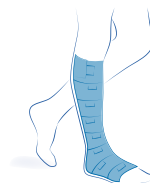

Half-square
Overlapping

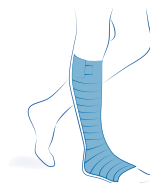

Full-square
Overlapping

The **maintenance** phase

As soon as a satisfactory volume reduction is achieved, the maintenance phase of the lymphedema starts.

Its role is to stabilize the volume of the reduced limb in the initial phase, or even to continue the volume reduction, but also to avoid relapses and enable the patient to recover normal mobility.

This phase is conducted first with elastic compression garments which will be renewed regularly under the supervision of a healthcare professional.



Worn daily from waking up to bedtime, day-time compression garments ensure both:

- **compression** aimed at easing the circulation of the lymph to the heart, thus preventing the formation of edema.
- **containment** preventing the edema from recovering more than a certain percentage of volume, irrespective of its intrinsic expansion force.

To optimize treatment efficiency, it is recommended to wear light compression device during rest and night-time.



MOBIDERM garments have been developed to mobilize the edema during rest and night-time. They offer an efficient, easy to use and comfortable alternative to self-bandage.⁽¹⁾

The maintenance phase also includes manual lymphatic drainage at a lesser frequency, as well as the learning of self-massaging and self-bandaging as needed. Lastly, it requires the practice of an adapted physical activity, skin cares and regular medical monitoring.

(1) Mestre – Interest of an auto-adjustable night-time compression sleeve (MOBIDERM Autofit) in maintenance phase of upper limb lymphedema: the MARILYN pilot RCT Support Care Cancer (25:2455-2462) – 2017 – Page 4

VENO ELEGANCE

RAL



Standard circular knit compression garments



Standard products
Faster availability

For which patients

VENO ELEGANCE armsleeves are intended for use in the treatment of lymphedema in maintenance phase following a reduction phase.

The product is recommended for patients presenting:

- Mild to moderate lymphedema⁽¹⁾
- Regular limb shape without skin folds⁽¹⁾

Principle of action

Compression: to help lymphatic circulation and prevent the formation of edema⁽²⁾

Characteristics

Composition⁽³⁾: polyamide microfiber, elastane

Color: beige 

Compression class: class 2 (23 - 32 mmHg)



(1) SFMV – Compression and lymphoedema of the lower limb Compression Interface Committee - March 2015 - Page 2

(2) SNITEM – Coupé - Compression and restraint: what are the differences?, History and Innovation in Lymphology Dec 2015 - Page 5

(3) Please refer to the Instructions for Use of the product for the full list of components

Models and Options

Armsleeve & Armsleeve with mitten

23 - 32 mmHg



DESIGNATION


Armsleeve with anti-slip

	Ref Short	Ref Long
Small	2002D221102561	2002D221102563
Medium	2002D221202561	2002D221202563
Large	2002D221302561	2002D221302563

Armsleeve with mitten and anti-slip

	Ref Short	Ref Long
Small	2002D221102551	2002D221102553
Medium	2002D221202551	2002D221202553
Large	2002D221302551	2002D221302553

Sizes (in cm)

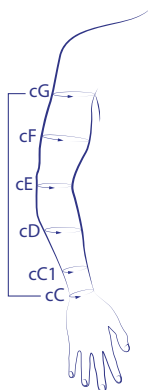

	SMALL	MEDIUM	LARGE
cG	25 - 29.5	30 - 35.5	36 - 42
cF	23 - 26.5	27 - 31	31.5 - 36
cE	21 - 24.5	25 - 28.5	29 - 33
cD	20 - 23.5	24 - 27.5	28 - 32
cC1	13.5 - 16	16.5 - 19	19.5 - 22.5
cC	13 - 15.5	16 - 18.5	19 - 22
Length SHORT			
LG	38 - 44	40 - 46	42 - 48
Length LONG			
LG	42 - 48	44 - 50	46 - 52

MOBIDERM autofit

Standard adjustable mobilizing garments

Description

The **MOBIDERM autofit** garments are designed to mobilize edematous or indurated subcutaneous tissues.

They are intended for the treatment of lymphedema in maintenance phase at rest and night to maintain the volume reduction.

The standard, self-adjustable garment adapts to most of patient's individual body shape and evolution overtime.

They are available in multiple models:

- For upper limb: **MOBIDERM autofit** armsleeve with mitten.
- For lower limb: **MOBIDERM autofit** below-knee or thigh-high.

For which patients ?

MOBIDERM autofit garments are recommended for patients presenting:


- Mild to moderate lymphedema
- Regular or non-regular limb shape
- Stable or evolutive edema



Principle of action

The **MOBIDERM** autoFit garments incorporate **MOBIDERM** technology that is made up of foam blocks encased between two non-woven bandages.

Due to the difference in pressure it creates between the support area of the blocks and their surrounding area, it helps stimulate the flow of lymphatic fluid and optimizes drainage efficacy⁽¹⁾.

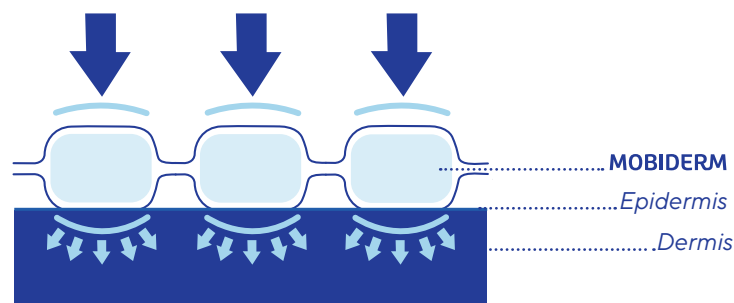

Characteristics

Composition⁽²⁾:

- Outer layer: polyester, polyamide, elastane
- Inner layer along the leg/arm: adhesive non-woven, polyurethane foam (15 x 15 mm blocks)
- Inner layer foot/hand level: adhesive non-woven, polyethylene foam (5 x 5 mm blocks)

Color: beige 

Learn how to fit Mobiderm autofit here!



Upper limb: <https://www.youtube.com/watch?v=gcTrtkPIR94>



Lower limb: <https://www.youtube.com/watch?v=zv2Qaj59614>

(1) Todd – Mobiderm Autofit: an adjustable sleeve that enables patients to self-manage lymphedema – Chronic edema – April 2018 – Page 6
(2) Please refer to the Instructions for Use of the product for the full list of components

MOBIDERM autofit

Standard adjustable mobilizing garments

Upper limb

Several tightening notches

- adaptation to most of patient's individual limb shape
- adjustment marks: precise and repeatable application, control of the edema volume variation

Self-gripping fasteners

- easy and quick set up⁽¹⁾


Semi-open garment

easy to put on, easy to take off⁽¹⁾

Sleeve with attached retractable mitten

- helps limiting the risk of edema migration
- the hand can be freed for usual tasks

Sizes

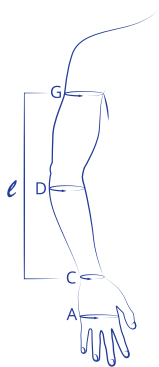

ℓ: Length from wrist to axillary
cA: Hand circumference
cC: Wrist circumference
cD: Forearm circumference (at the widest point)
cG: Axillary arm circumference

Size	cA (in cm)	cC (in cm)	cD (in cm)	cG (in cm)	Reference Normal version ℓ = 40 - 45 cm	Reference Long version ℓ = 45 - 50 cm
Right armsleeve						
1	17 - 21	15 - 19	22 - 28	25 - 32	373302 20100261	373302 20100271
2	17 - 21	16 - 20	24 - 30	29 - 36	373302 20200261	373302 20200271
3	18 - 22	17 - 21	26 - 32	33 - 40	373302 20300261	373302 20300271
4	18 - 22	17 - 21	28 - 34	37 - 44	373302 20400261	373302 20400271
5	19 - 23	18 - 22	30 - 36	41 - 48	373302 20500261	373302 20500271
6	20 - 24	19 - 23	31 - 37	45 - 52	373302 20600261	373302 20600271
Left armsleeve						
1	17 - 21	15 - 19	22 - 28	25 - 32	373302 20100262	373302 20100272
2	17 - 21	16 - 20	24 - 30	29 - 36	373302 20200262	373302 20200272
3	18 - 22	17 - 21	26 - 32	33 - 40	373302 20300262	373302 20300272
4	18 - 22	17 - 21	28 - 34	37 - 44	373302 20400262	373302 20400272
5	19 - 23	18 - 22	30 - 36	41 - 48	373302 20500262	373302 20500272
6	20 - 24	19 - 23	31 - 37	45 - 52	373302 20600262	373302 20600272

(1) Mestres - Interest of an auto-adjustable night-time compression sleeve (MOBIDERM Autofit) in maintenance phase of upper limb lymphedema: the MARILYN pilot RCT - Support Care Cancer (25:2455-2462) - 2017 - Page 6

Lower limb



Several tightening notches

- adaptation to most of patient's individual limb shape
- adjustment marks: precise and repeatable application, control of the edema volume variation



Semi open garment

- easy to put on, easy to take off⁽¹⁾



Self-gripping fasteners

- easy and quick set up⁽¹⁾



Double row


- wide adaptation range
- allows right or left leg use



Anti-slip sole

- limits slippage when walking

Sizes

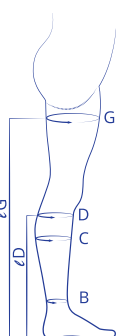

Below-knee

Size	cB (in cm)	cC (in cm)	cD (in cm)	Reference (right/left)			
				Short ℓD = 38 - 40 cm	Normal ℓD = 40 - 42 cm	Long ℓD = 42 - 44 cm	Xtra Long ℓD = 44 - 46 cm
1	16 - 24	24 - 35	25 - 36	373401 20100231	373401 20100232	373401 20100233	373401 20100234
2	20 - 28	32 - 43	32 - 43	373401 20200231	373401 20200232	373401 20200233	373401 20200234
3	24 - 32	40 - 51	39 - 50	373401 20300231	373401 20300232	373401 20300233	373401 20300234
4	28 - 36	48 - 59	46 - 57	373401 20400231	373401 20400232	373401 20400233	373401 20400234

Thigh-high

Size	cB (in cm)	cC (in cm)	cG (in cm)	Reference (right/left)			
				Short ℓG = 67 - 72 cm	Normal ℓG = 72 - 77 cm	Long ℓG = 77 - 82 cm	Xtra Long ℓG = 82 - 87 cm
1	16 - 24	24 - 35	46 - 57	373501 20100231	373501 20100232	373501 20100233	373501 20100234
2	20 - 28	32 - 43	53 - 65	373501 20200231	373501 20200232	373501 20200233	373501 20200234
3	24 - 32	40 - 51	61 - 72	373501 20300231	373501 20300232	373501 20300233	373501 20300234
4	28 - 36	48 - 59	68 - 80	373501 20400231	373501 20400232	373501 20400233	373501 20400234

ℓD: Floor-to-knee height (measurement taken below the bend of the knee, back side)
ℓG: Floor-to-thigh height (measurement at the gluteal fold)
cB: Ankle circumference (at the narrowest point)

cC: Calf circumference (at the widest point)
cD: Below knee circumference (on the head of the fibula)
cG: Thigh circumference (at the widest point)

STANDARD MOBIDERM GARMENTS

Standard mobilizing garments

Description

The **MOBIDERM** standard garments are designed to mobilize edematous or indurated subcutaneous tissues. They are intended for the treatment of lymphedema in maintenance phase at rest and for night to maintain the volume reduction. They are available in 3 models: armsleeve, mitten or glove.

For which patients ?

MOBIDERM standard garments are recommended for patients presenting:

- Mild to moderate lymphedema
- Regular limb shape
- Stable lymphedema volume

Characteristics

Composition⁽¹⁾:

- Outer layer: polyamide, elastane
- Armsleeve inner layer: adhesive non-woven, polyurethane foam (15 x 15 mm blocks)
- Mitten and glove inner layer: adhesive non-woven, polyethylene foam (5 x 5 mm blocks)

Color: beige 




Armsleeve

Size	cC (in cm)	cE (in cm)	cG (in cm)	Reference (right/left)	
				Normal ℓ = 40 - 45 cm	Long ℓ = 45 - 50 cm
1	14 - 16	22 - 25	25 - 29	373002 20100232	373002 20100233
1+		24 - 27	28 - 32	373002 40100232	373002 40100233
2	16 - 18	25 - 28	29 - 33	373002 20200232	373002 20200233
2+		27 - 30	31 - 35	373002 40200232	373002 40200233
3	18 - 20	28 - 31	32 - 36	373002 20300232	373002 20300233
3+		30 - 33	35 - 39	373002 40300232	373002 40300233
4	20 - 22	31 - 34	35 - 39	373002 20400232	373002 20400233
4+		33 - 36	38 - 42	373002 40400232	373002 40400233
5	22 - 24	34 - 37	38 - 42	373002 20500232	373002 20500233
5+		36 - 39	41 - 45	373002 40500232	373002 40500233



A: Hand circumference
(head of metacarpal bones)
C: Wrist circumference
E: Elbow circumference
G: Axillary arm circumference

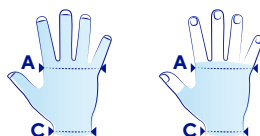

Glove

Size	cC (in cm)	cA (in cm)	Reference	
			right	left
1	14 - 16	15 - 17	373201 20100201	373201 20100202
2	16 - 18	17 - 19	373201 20200201	373201 20200202
3	17 - 19	19 - 21	373201 20300201	373201 20300202
4	19 - 21	21 - 23	373201 20400201	373201 20400202
5	20 - 22	23 - 25	373201 20500201	373201 20500202
6	21 - 23	25 - 27	373201 20600201	373201 20600202

Mitten

Size	cC (in cm)	cA (in cm)	Reference	
			right/left	
1	14 - 16	15 - 17	373101 20100299	
2	16 - 18	17 - 19	373101 20200299	
3	17 - 19	19 - 21	373101 20300299	
4	19 - 21	21 - 23	373101 20400299	
5	20 - 22	23 - 25	373101 20500299	
6	21 - 23	25 - 27	373101 20600299	

(1) Please refer to the Instructions for Use of the product for the full list of components.

MADE-TO-MEASURE PATIENT MOBIDERM GARMENTS



Description

The **MOBIDERM** made-to-measure patient garments are designed to mobilize edematous or indurated subcutaneous tissues.

They are intended for the treatment of lymphedema in maintenance phase for day and night to maintain the volume reduction.

They are available in multiple models:

- For upper limb: armsleeve ①, armsleeve with mitten and without thumb ② or armsleeve with mitten and thumb ③
- For lower limb: below-knee ④ or thigh-high ⑤

For which patients

MOBIDERM made-to-measure patient garments are recommended for patients presenting:

- Moderate to severe lymphedema
- Non-regular limb shape
- Out of standard sizes
- Stable lymphedema volume

Characteristics

Composition⁽¹⁾:

- Outer layer: polyamide, elastane
- Inner layer: adhesive non-woven, polyurethane foam (15 x 15 mm blocks) or polyethylene foam (5 x 5 mm blocks)

Color: beige



Lymphedema Treatment:
the reduction phase

Lymphedema Treatment:
the maintenance phase

Venoflex for early venous Insufficiency

Venoflex for advanced venous
Insufficiency & lymphoedema

MOBIDERM INTIMATE BRA

For which patients

The Mobiderm Intimate Bra is recommended for patients presenting:

- Acute or chronic breast and/or chest wall edema
- Chronic infiltration after reconstruction

Benefits

- **Ready to use:** compressing lymph bra with mobilising Mobiderm pads (2 pieces) that are integrated, removable and ready for immediate use
- **Self management:** easy to put on, hook and zip front closure, softly lined integrated pockets for breast form
- **Discreet color:** light nude color, not visible even under light clothes

Features

- **Adaptable to most body shapes:** several sizes available
- **Flexible and thin:** thanks to the physical properties of the materials: polyamide, elastane⁽¹⁾
- **A turnkey solution:** bra + pad
- **Possibility of inserting the pad** (or an external breast prosthesis) inside the bra or removing it **according to the patient's needs**
- **Easy to put on and take off:** thanks to the opening on the front of the bra

Color: nude



Compression class

- Light compression 5-10 mmHg



(1) L'atelier du papetier - Les fibres textiles

Chest

The bra is intended for women who have had breast cancer surgery, to reduce swelling and prevent the appearance of lymphedema.

After breast plastic surgery, the Mobiderm Intimate bra has a form-stabilizing effect⁽¹⁾ and provides support through compression and drainage of edema⁽²⁾⁽³⁾ that may develop after breast cancer treatment.

- The bra exerts increased pressure⁽⁴⁾ on the tissue areas affected by edemas. This improves lymphatic drainage⁽³⁾.
- Compression in the scar tissue area minimizes the development of hematomas⁽⁵⁾.



Front view



Rear view

Sizes

Underbust size INT	Size	68-72 70	73-77 75	78-82 80	83-87 85	88-92 90	93-97 95	98-102 100	103-107 105	108-112 110	113-117 115
Cup size		XS		S		M		L		XL	
	AA / A	80 - 89 T571301201AAA24		90 - 99 T571301202AAA24		100 - 109 T571301203AAA24		112 - 119 T571301204AAA24		120 - 129 T571301205AAA24	
	B / C	84 - 93 T5713012010BC24		94 - 103 T5713012020BC24		104 - 113 T5713012030BC24		116 - 123 T5713012040BC24		124 - 133 T5713012050BC24	
	D / E	88 - 97 T5713012010DE24		98 - 107 T5713012020DE24		108 - 117 T5713012030DE24		120 - 127 T5713012040DE24		128 - 137 T5713012050DE24	



Discover the video tutorial with donning instructions for Mobiderm Intimate Bra on our **Thuasne YouTube channel!**

(1) Due to its origin; the Mobiderm Intimate bra is based on the Cara compression bra.

(2) Thanks to the pad.

(3) I. Quéré, E. Presles, M. Coupé, S. Vignes, L. Vaillant, D. Eveno, S. Laporte, A. Leizorovicz, Investgateurs étude POLIT – Prospective multicenter. Observational study of lymphoedema therapy: POLIT study – Journal des maladies vasculaires 2014) 39, 256-263

(4) Refer to CE marking

(5) Refer to CE marking data

MOBIDERM INTIMATE SHORTS

Compression and mobilization shorts
Treatment of pelvic and genital lymphedema

For which patients

Mobiderm Intimate shorts is intended for patients with:

- Pelvic and/or genital edema
- Mild to severe lymphedema
- Stable or progressive edema
- Regular or irregular morphology



Benefits



- Improve the quality of life*: can be worn throughout the day and night and the product is discreet
- Easy to use: the product is ready to wear, all-in-1 product with removable pad, and it's easy to use for 3 out of 4 patients*

Composition

- **Compression shorts:** polyamide, elastane, polyester
- **Mobiderm mobilizing genital pad small blocks:** polyester, polyamide, acrylic glue, polyethylene foam

Sizes

		cK (cm)	cG (cm)	
	cK			
	cG			
	1	88 - 94	45 - 54	
	2	95 - 101	48 - 58	
	3	102 - 108	51 - 62	
	4	109 - 115	54 - 66	
	5	116 - 122	57 - 69	
	6	123 - 129	60 - 72	

		cK (cm)	cG (cm)	
	cK			
	cG			
	1	88 - 94	45 - 54	
	2	95 - 101	48 - 58	
	3	102 - 108	51 - 62	
	4	109 - 115	54 - 66	
	5	116 - 122	57 - 69	
	6	123 - 129	60 - 72	

cK (cm) : Pelvic circumference
cG (cm) : Upper thigh circumference



Pelvic and genital area



Men

Features:

Conform:

- Soft fabric⁽¹⁾⁽²⁾ for more comfort
- Flat seams to limit skin reactions

Easy donning⁽³⁾:

Easy to put in and out – elastic fabrics

Easy measuring: only 2 measurements to take and only one size table.

Mobiderm pad:

- Covered with soft⁽¹⁾⁽²⁾ fabric for more comfort
- Small blocks for more adaptability^{(4)*}
- 2 Mobiderm pads versions to adapt various anatomies:
1 men version, to be placed in the pocket and around the testicles
1 women version, to be placed in the shorts inner pocket

Color: Dark blue/Black



Women



- (1): **CC1508** - Mobiderm Panty - Validation Plan (2019)
 (2): **CC1508** - Rational formulation (2021)
 (3): **CC1508** - Usability Test Report - Summative Evaluation (2020)
 (4): Please refer to internal EC data
 * The contact surface is reduced, optimisation of the number of blocks.



Discover all available video tutorials Mobiderm Intimate fitting instructions **on our Thuasne YouTube channel!**

* OLYMPY study: Exploratory study to assess the clinical benefits of wearing the MOBIDERM Intimate Short in patients with pelvic and/or genital lymphedema. - 2021 32 patients - n° ID-RCB: 2020-A00247-32 ; primary objective: assessment of the effect of the MOBIDERM Intimate short, after 12 weeks, on an overall impression of change in their quality of life in relation to PL/GL

VENOFLEX

VENOFLEX devices are designed to be used on the lower limbs by adults meeting the indications outlined below. They provide compression, applying pressure to the limb and thereby aiding venous return.

INDICATIONS:

- Restless legs
- Heavy legs
- Reticular varicose veins
- Telangiectasia
- Prevention during standing, prolonged standing
- Long journeys
- Evening oedema
- Telangiectasia or reticular veins (< 3 mm)
- Following sclerotherapy for telangiectasia or reticular veins (< 3 mm)
- Following sclerotherapy or surgery for varicose veins (≥ 3 mm)
- Varicose veins (≥ 3 mm)
- Prevention of venous thrombosis in pregnant women or post-partum

Compression level: 15 – 20 mmHg




FOR HER VENOFLEX KOKOON ABSOLU

Comfort and softness for cocooning moments⁽¹⁾

Description

- The semi-opaque range
- Broadest Venoflex palette of sizes and colours
- Soft mesh in opaque microfiber
- Flat seams
- Reinforced heel and foot arch
- Instep shaping
- Anatomic calf shape
- Machine washable

For which patients?


- Those who desire an opaque finish to their hosiery
- Casual style
- Those requiring a flexible and soft material 



FOR HIM VENOFLEX FASTCOTON

Absorption and evacuation of moisture⁽²⁾⁽³⁾

Description

- Solidity and durability thanks to cotton* fabrication process (twisted thread).
- Ease of use: extensible widthwise and heightwise & easy to put on and take off.
- Comfort: absence of edge.
- Simplicity: easy to care for.
- Technicity: contemporary colours & seams on top and not on the tip of the foot.
- Machine washable 

For which patients?

- Those who wish to have a product made of natural fibers* for every day
- Casual style
- Those wishing to wear socks that are easy to put on and comfortable**



* Composition: polyamide, elastane, cotton. ** Extensible widthwise and heightwise.

(1) Soft mesh composition. (2) Hygrothermal test IFTH n°12047A realised in July 2012. (3) Measurement taken on the upper part of the sock stretched to the application measurements.

VENOFLEX

VENOFLEX devices are designed to be used on the lower limbs by adults meeting the indications outlined below. They provide compression, applying pressure to the limb and thereby aiding venous return.

INDICATIONS:

- Varicose veins (≥ 3 mm)
- Following sclerotherapy or surgery for varicose veins (≥ 3 mm)
- Chronic venous oedema
- Trophic disorder (pigmentation, eczema, venous lipodermatosclerosis or atrophie blanche)
- Healed venous ulcer
- Chronic venous disease associated with pregnancy or the post-partum period
- Prevention and/or treatment of venous thrombosis (lower limb) and post-thrombotic syndrome
- Lower limb lymphoedema (maintenance phase)

Compression level: 20 - 36 mmHg

INDICATIONS:

- Primary and secondary varicosis, varices during pregnancy, after sclerotherapy or venous surgery.
- Superficial thrombophlebitis and condition after healed phlebitis, deep vein thrombosis, condition after thrombosis, post thrombotic syndrome, thrombosis prophylaxis.
- Chronic venous insufficiency (CVI) of stages I to III according to Widmer and C1S-C6 according to CEAP, ulcer prevention, venous insufficiency, angiodysplasia.
- Lymphedema, edema in pregnancy, post-traumatic edema, postoperative edema, lipoedema, lipo-lymphedema, phlebo-lymphatic edema, congestive conditions as a result of immobility (arthrogenous congestion syndrome, pareses and partial pareses of a limb).

Compression level: 23 - 32 mmHg




FOR HER

VENOFLEX KOKOON ABSOLU

Comfort and softness for cocooning moments⁽¹⁾

Description

- The semi-opaque range
- Broadest Venoflex palette of sizes and colours
- Soft mesh in opaque microfiber
- Flat seams
- Reinforced heel and foot arch
- Instep shaping
- Anatomic calf shape
- Machine washable 

For which patients?

- Those who desire an opaque finish to their hosiery
- Casual style
- Those requiring a flexible and soft material



Lymphedema Treatment:
the reduction phase

Lymphedema Treatment:
the maintenance phase

Venoflex for early venous Insufficiency

Venoflex for advanced venous
Insufficiency & lymphoedema

(1) Soft mesh composition

WINGS FOR YOUR HEALTH

Indications:

MOBIDERM pad & bandages: treatment of acute or chronic edema (for example: lymphedema). **MOBIDERM** garments: maintenance of the reduction in lymphedema swelling. **MOBIDERM INTIMATE:** treatment of pelvic and genital lymphedema. **MOBIDERM autofit:** maintenance of the reduction in lymphedema swelling. **COTTON BANDAGE:** For skin protection and immobilisation in multicomponent bandaging. **BIFLEXIDEAL:** Treatment of edema (traumatic or chronic). Treatment of lymphedema. Chronic venous insufficiency (as an alternative to compression stockings). Following sclerotherapy or varicose vein surgery. Treatment of venous ulcers (with ABI > 0.8 or under close medical supervision for an ABI between 0.5 and 0.8). Symptomatic treatment of deep or superficial acute venous thrombosis. Prevention of post-thrombotic syndrome: alternative to compression stockings for at-risk patients; Alternative to compression stockings for the treatment of edema of the legs or swelling during pregnancy. Treatment of symptoms related to soft tissue injuries. **BIFLEX:** Alternative to compression stockings (15-36 mmHg) for varicose veins. Chronic edema. Post-traumatic edema. After sclerotherapy or surgery of varicose veins as an alternative to compression stockings. Alternative to compression stockings for trophic disorders (dermatitis/lipodermatosclerosis). Venous leg ulcers (multitype bandaging, elastic bandage for patients with reduced mobility). Symptomatic treatment of acute superficial and deep venous thrombosis. Alternative to compression or antithrombotic stockings. Post-thrombotic syndrome prevention: alternative to compression stockings for high-risk patients. Lymphedema treatment. **BIPLAST:** Soft tissue or musculoskeletal injuries. Taping or strapping. Fracture of small bone (finger, foot, etc.). Bone deformity. Vascular or lymphatic conditions requiring short-stretch or multicomponent bandaging. Biplast bandages can be used to secure dressings and splints. **BIFLEX+ GRADUATED:** Alternative to compression stockings (15-36 mmHg) for varicose veins. Chronic edema. Post-traumatic edema. After sclerotherapy or surgery of varicose veins as an alternative to compression stockings. Alternative to compression stockings for trophic disorders (dermatitis/ lipodermatosclerosis). Venous leg ulcers (multitype bandaging, elastic bandage for patients with reduced mobility). Symptomatic treatment of acute superficial and deep venous thrombosis. Alternative to compression or antithrombotic stockings. Post-thrombotic syndrome prevention: alternative to compression stockings for high-risk patients. Lymphedema treatment. **VENO ELEGANCE:** Soft indentable lymphedema (stage I), arm edema following breast cancer surgery and radiotherapy, post-surgery and post-traumatic edema. Contraindications: Advanced peripheral artery occlusive disease, decompensated cardiac insufficiency, septic phlebitis, phlegmasia coerulea dolens, weeping dermatoses, compression material intolerance, paresthesia, advanced peripheral neuropathy, primary chronic polyarthritis. **MOBIDERM INTIMATE BRA:** Acute or chronic edemas of the breast and/or the breast wall. Post-operative edemas or edemas after radio therapy (only on intact skin). Chronic infiltration after reconstruction, remaining chronic breast wall edemas. **MOBIDERM INTIMATE SHORT:** Treatment of pelvic and genital lymphedema. **VENOFLEX KOKOON ABSOLU & FAST COTON:** 15-20 mmHg/Class 2: heavy legs, restless legs, evening oedema, telangiectasia or reticular veins (< 3 mm), following sclerotherapy for telangiectasia or reticular veins (< 3 mm), following sclerotherapy or surgery for varicose veins (≥ 3 mm), varicose veins (≥ 3 mm), prevention of venous thrombosis in pregnant women or post-partum, prevention of venous thrombosis during long journeys, treatment of lymphoedema (maintenance phase). • 20-36 mmHg/Class 3: varicose veins (≥ 3 mm), following sclerotherapy or surgery for varicose veins (≥ 3 mm), chronic venous oedema, trophic disorders (pigmentation, eczema), chronic trophic disorders (lipodermatosclerosis or atrophie blanche), healed venous ulcer, chronic venous disease associated with pregnancy or the post-partum period, prevention and/or treatment of venous thrombosis (lower limb) and post-thrombotic syndrome, lower limb lymphoedema (maintenance phase).

Availability of these products might vary from a given country or region to another, as a result of specific local regulatory approval or clearance requirements for sale in such country or region. The medical devices mentioned on this document are CE marked according to the Regulation 2017/745 on medical devices. Products mentioned in this document are CE class I devices (except for the cotton bandage which is a Medical Device accessory). Please contact Thuasne should you need any additional information on devices classification. Please read carefully the instructions for use, indications and contraindications of the product. We reserve the right to modify our products as shown and described, without previous notice. Non-contractual photographs. Please check the availability of products according to the countries. Ref: 2303119 - Last revision date: 04-2023. Photos : Studio Caterin - Krakenimages/Shutterstock.com.



Thuasne France
120 rue Marius Aufan
92300 Levallois-Perret Cedex - France
www.thuasne.com
sales.export@thuasne.fr

Thuasne Deutschland
Im Steinkamp 12,
30938 Burgwedel
Deutschland

PROUDLY DISTRIBUTED
IN NEW ZEALAND BY

medivex
HEALTHCARE LIMITED

Medivex Healthcare Ltd
335 Onehunga Mall,
Onehunga, Auckland 1061
0800 486 483
sales@medivex.co.nz
medivex.co.nz